



Using eSlide Manager and WebViewer


Visit <http://eslide.osumc.edu> and enter the ABOMP login credentials.

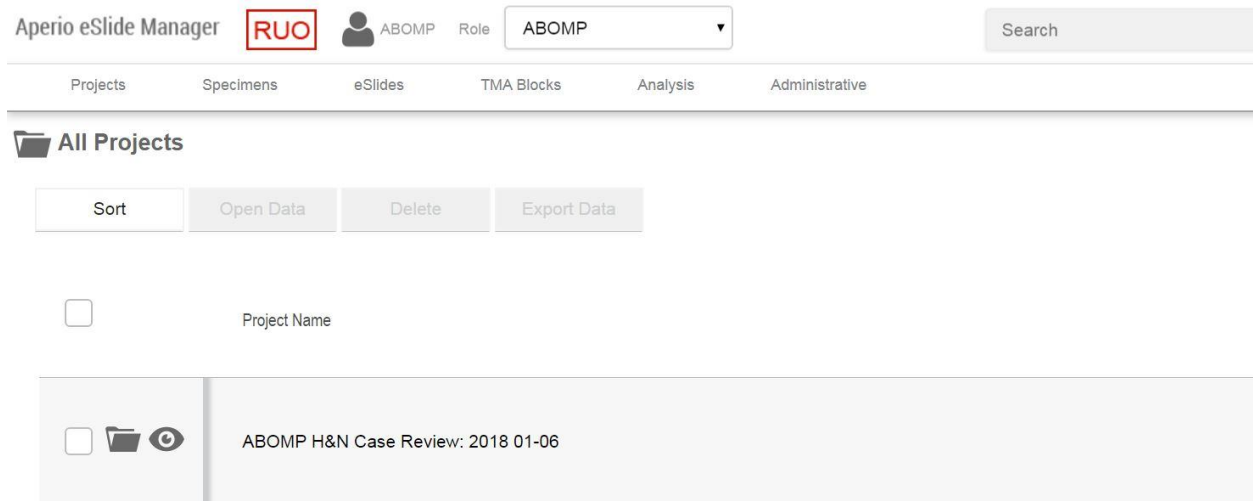

Click the **Eye** icon next to the desired **Project Name**.

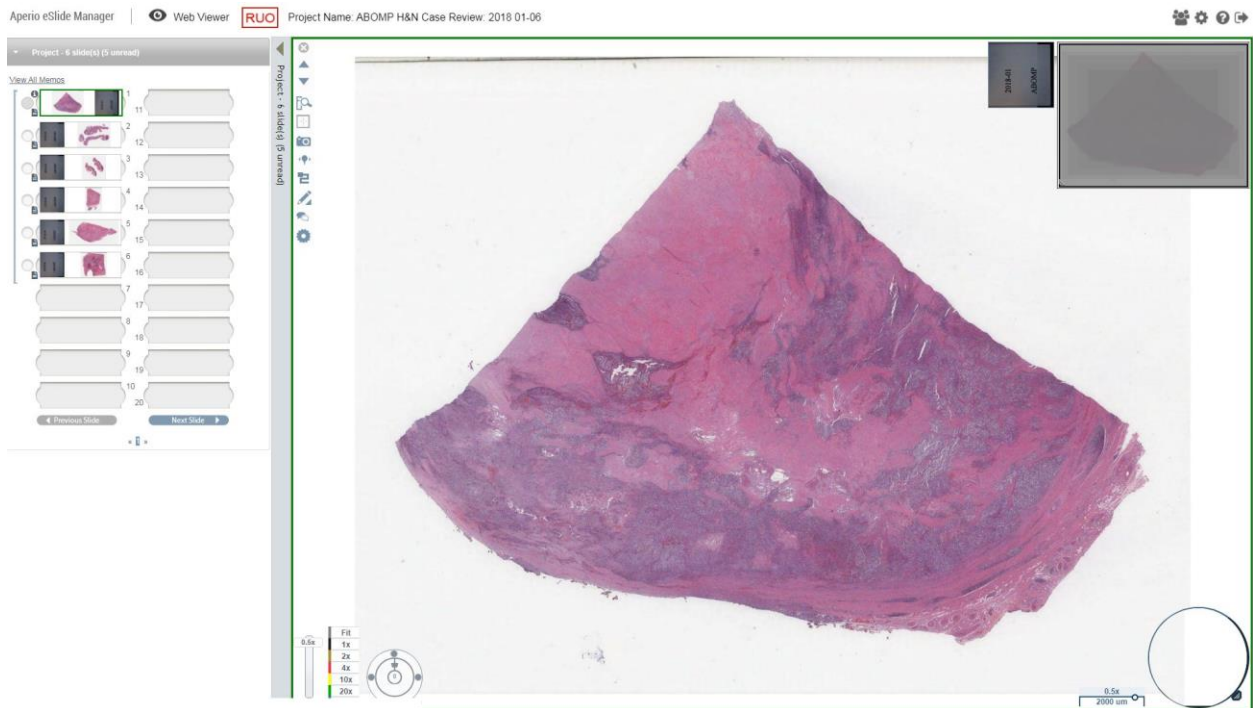

The project will open in **ImageViewer** displaying all **WSI** contained within organized by groups of 20 similar to a glass slide tray.

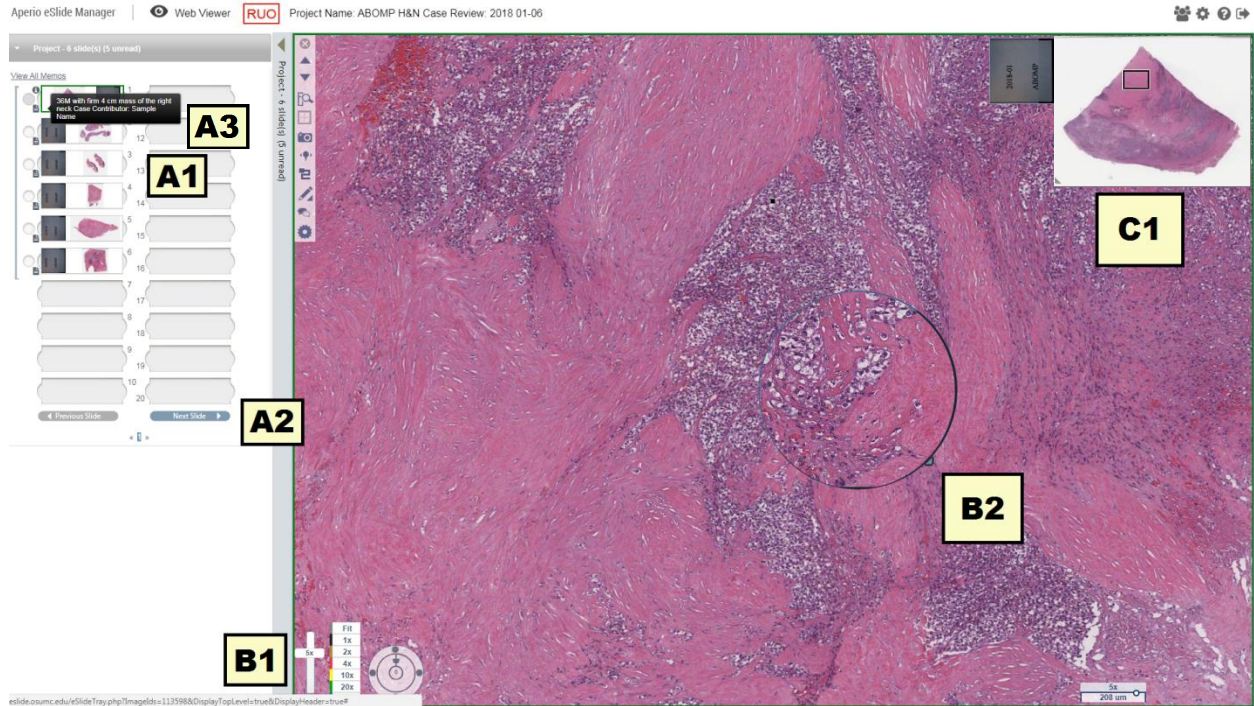

Open the desired WSI either clicking on the **Thumbnail (A1)** or the **Next Slide** icon (**A2**). Click on the **Document** icon next to each thumbnail for Case History (**A3**).

Read WSI are indicated by thumbnails rotating their orientation from slide label on the left to slide label on the right. Right click on thumbnail to manually toggle between Read/Unread.

Use mouse wheel to select change objective or use the **Zoom Control / Zoom Slider (B1)**. Manually pan by left clicking and dragging mouse through WSI. Magnifying Lens (**B2**) can be moved and enlarged.

Use the image thumbnail to track your whereabouts within the selected WSI (**C1**).